Pinze per Allineatori • Aligners Pliers

ABC LINE

D30-1015

PINZA PER TAGLIO A CERCHIO ROUND HOLE FORMING PLIER

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Plier to make round cuts to create room for other devices (tractions such as lingual buttons, eyelet, TS2). To create a space on a specific tooth, place the plier on the lingual side of the aligner, then press to cut.

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Plier to make a drop cut, useful in treatments where the use of the elastics is required. Place the plier on the aligner's edge and press sufficiently to cut it. The cut created works as an anchor for the application of the elastics directly on the aligner.

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Plier to make a rectangular pressure point on the aligner's mesial or distal side. Place the instrument, mesially or distally, where you want to make the pressure point, and press up until the shape is done. Use the special key to regulate the depth of the pressure point.

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Plier to make cylindrical pressure points on the aligner's mesial or distal side. Place the instrument, mesially or distally, where you want to make the pressure point, and press up until the shape is done. Use the special key to regulate the depth of the pressure point.

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Plier to make bite plans on the aligner. Use the special key to regulate the depth of the bite plan. Heat the plier on a flame before using it. Once heated, place it on the lingual side of the aligner and press to form the bite plan.

i-liner

think different

Ed. 05/2022

SIA
ORTHODONTIC
MANUFACTURER

SIA Orthodontic Manufacturer srl
Zona Industriale snc, Rocca d'Evandro
81050, Caserta, Italy

 +39 0823 908029

 info@i-liner.it

 www.i-liner.it