## Congratulations on your decision

You are on your way to a wonderful smile. Each one of your i-liner was custom-made especially for you. Please follow the suggestions in this guide to ensure successful treatment and proper care for your i-liner.

### + Wearing your i-liner

We suggest you try to wear the i-liner all the time except while eating, drinking hot drinks, and brushing your teeth. It will take about one to two weeks for you to acclimate to the appliance completely. You may experience some difficulties with speech and/or might salivate more during this period of acclimation. During the course of treatment, you will feel one or two teeth a bit tender or under pressure as they are moved to a new position. The longer you wear the appliance per day, the shorter your treatment time.

### + Avoid the Following

1. Never expose your i-liner to heat; heat will distort the appliance, and it will not fit correctly.
2. Never boil it to clean or sterilize the i-liner.
3. Never leave the appliance in your car in the summertime.
4. Never wrap it in a napkin when you take the Aligner out to eat. You risk losing it or throwing it away, not realizing that you left the i-liner in the napkin.
5. If the appliance causes you the discomfort of any sort, contacts your dentist, do not try to modify or "repair" the appliance by yourself.

### + Cleaning your i-liner

It is important to keep the appliance clean, so it remains clear and odor-free. The best way to clean the appliance is to use an ultrasonic cleaner and a gentle cleaning solution such as BiteWash

You may also use a retainer or denture cleaner found at your local store (follow product instructions) and a soft toothbrush.

### + Important Tips

1. Store your i-liner in the provided case when it is not in your mouth.
2. Clean the i-liner daily using a safe cleaner and method.
3. Wear the i-liner every day as per your dentist's instructions.
4. Always use your fingers to place the appliance in your mouth; never use your teeth to "bite" the appliance into place. Using your teeth will lead to distortion or breakage of your i-liner.

**Your dentist's instructions supersede these suggestions.**

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